



Pharmacy Refrigerators

B Medical Systems | P Range

Pharmacy Refrigerators are devices intended for the safe storage of pharmaceuticals, e.g. medications or live virus vaccines, at cold temperatures between +4°C and +15°C.

Compliant to DIN 13221 | DIN 58345 | ÖNORM K 2040 | Medical Device according to MDR (EU) 2017/745, Class I or 21CFR Part 862.2050, Class I













SAVING LIVES
THROUGH RELIABLE
AND INNOVATIVE
TECHNOLOGY

Medical Refrigeration





Safety Standards **B Medical Systems**











The Safety Standards developed by B Medical Systems define certain significant technical features of a product. These ensure the safe storage of the preparations as well as setting the highest standards of safety for the user.

PRECISION LINE	PREMIUM LINE	
		B Medical Systems Electronics
		B Medical Systems digital display
		B Medical Systems 7" full touchscreen display
		Safety door lock (with 2 keys) and key-operated power switch ON/OFF (with 2 keys)
		Power indicator light and digital temperature indicator (display: 0.1 digits)
		Controlled fan cooling system for constant temperature and even temperature distribution across the entire refrigerating chamber [not applicable for model PF260 (Freezer compartment)]
		Automatic switch-off of the evaporator fans when the door opens [not applicable for model PF260 (Freezer compartment)]
		Self-contained alarm system with integrated battery takes over the alarm function and temperature value measurements in case of system failure for at least 48 hours
		Acoustic and visual alarm signal in case of temperature alarm and system failure
		The alarm history function on the electronic stores all the relevant values during a temperature alarm, such as: min., max., average temperature and alarm duration
		Remote transmission alarm signal (via potential-free contact) in case of temperature alarm (change-over contact)
		Safety thermostat prevents dropping of the temperature below +2°C
		Door opening alarm (visual / acoustic)
		Designed and tested for climatic class SN (ambient temperature range +10°C to +32°C)
		Designed and tested for climatic class SN / T (ambient temperature range +10°C to +43°C)
		Interior made from stainless steel
		Internal LED lighting
		Additional remote transmission alarm signal (via potential-free contact) in case of system failure (change-over contact)
		Smooth castors for optimum flexibility of movement
	\circ	Ambient temperature sensor
\circ	\circ	B Connected - Universal software for the monitoring of refrigeration devices, including the acquisition, recording and visualization of temperature data
		DCU (Data Communication Unit) - Hardware module monitoring all operating conditions and passing them through to a central database – via local network (in combination with °B Connected)
		Ethernet interface for the visualization of all operating and control functions (hardware and software settings) via B Connected monitoring software on a peripheral device (computer)

Pharmacy Refrigerators

B Medical Systems | P Range

9 models • Volume 52 > 895 L • Set temperature +5°C • Climate class SN | SN/T • Compliant to DIN 13221 | DIN 58345 | ÖNORM K 2040 | MDR (EU) 2017/745, Class I

In conformity with national and international guidelines, regulations for Medical Devices offering reliability, efficiency and safety at an optimal price.

AIC W

PF260 | PRECISION LINE

MODELS P55-130-290-380 &

• Integrated multifunction monitoring electronics

Integrated B Medical Systems multifunction electronics; with easy access in the door level; offers alarms, central alarm system, set point security, battery backup, full display and many other control features; compatible with "B Connected solution for data monitoring and recording.



Lifetime warranty corrosion-free cabinet

- One piece material cabinet with a lifetime warranty for zero corrosion, highest quality polypropylene for better insulation and no sharp edges to provide easier maintenance and cleaning
- Smart ergonomy Easier and safer handling because of heavier components placed at the bottom



High storage capacity

High storage per square meter in terms of net volume, with a very versatile inner volume utility due to multiple user friendly rails allowing modularity.




Exclusive integrated electronics handle bar (Patent pending)


Pharmacy Refrigerators are devices intended for the safe storage of pharmaceuticals, e.g. medications

of expertise in refrigeration technology.

or live virus vaccines, at cold temperatures between +4°C and +15°C with multiple controls and warning safety features developed as a result of 40 years

All functionalities are easily accessible:

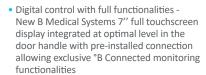

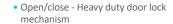



High quality materials

New high quality coating, certified medical devices quality and antibacterial, high quality steel for better longevity and easy hygiene control




Versatile modularity

Excellent storage capacity and modularity - Large choice of shelves and drawers offering modularity for every need.



4-layer (Precision Line) and 5-layer (Premium Line) PMMA organic glass offering highest insulation, combined with reduced weight reducing efforts to close door and constraints on hinges, offering better longevity.

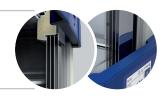





Technical DataGeneral features











		F	P 55	P	130	P	290	P	380	PF	-260	Refrigerator combined	& Freezer	
Gro	Gross / Net volume (I)			121 / 106		297 / 269		362 / 329		R 121 / 1	.06	F 121 / 1	06	
Set temperature (preset)			+5°C									-32°C		
Set temperature (setting range) can be adjusted in steps of 0.1℃			+4°C to +15°C									-32°C to -20°C		
Preset cold / warm alarm limit			+3°C / +7°C										-37°C / -27°C	
Hol	d over time	1.3 h (+5°C	1.3 h (+5°C to +10°C) 1.5 h (+5°C to +10°C)			1.0 h (+5°C to +10°C) 0.9 h (+5°C to +10°C)			1.5 h (+5°C	to +10°C)	0.7 h (-32°C	to -23°C)		
Clin	nate class (ambient temperature range)					SN (+10	°C to +32°C)							
Def	rosting technique		Natural										Manual	
Refrigerant type						F	R600a					R290		
Ext	ernal dimensions H x W x D (mm)	670 x 495	x 575	830 x 595 x 695		1670 x 595 x 695		1980 x 595 x 695		1650		x 595 x 695		
Inn	er dimensions H x W x D (mm)	470 x 375	x 350	630 x 475 x 470		1470 x 475 x 470		1790 x 475 x 470		630 x 475 x 470		630 x 475 x 470		
Net	weight with standard equipment (kg)	38		60		95		109				117		
	Supply voltage (V)	220-240	115-127	220-240	115-127	220-240	115-127	220-240	115-127	220-240	115-127	220-240	115-127	
	Frequency (Hz)	50 / 60	60	50 / 60	60	50 / 60	60	50 / 60	60	50 / 60	60	50 / 60	60	
v Sar	Power (W)	100	100	200	200	200	200	200	200	200	200	300	360	
	Energy consumption (kWh/24h)	0.7 / 0.8	0.8	1.0 / 1.0	0.9	1.8 / 1.8	1.9	2.0 / 2.0	2.0	1.0 / 1.0	0.9	2.8 / 2.9	3.1	
	Heat emission (Kcal/h)	25	29	36	32	64	68	72	72	36	32	100	111	
	Compressor running time (%)	31	31	24	19	35	40	32	40	24	19	51	50	
	Noise level (dB(A)) (at 1m height & 1m distance)	35 / 36	37	35 / 36	38	38 / 40	39	40	38	35 / 36	38	45 / 47	47	



Technical DataGeneral features









		P	400	PS	500	P	700	PS	900		
Gross / Net volume (I)		451 / 384		598 / 510		747 / 636		895 / 763			
Set	temperature (preset)	+5°C									
	temperature (setting range) be adjusted in steps of 0.1°C	+4°C to +15°C									
Pre	set cold / warm alarm limit				+3°	C / +7°C					
Hol	d over time				1.3 h (+5°C to +10°C)					
Clir	nate class (ambient temperature range)				SN / T (+1	L0°C to +43°C)					
Dej	frosting technique				N	atural					
Refrigerant type		R600a									
External dimensions H x W x D (mm)		1988 x 699 x 1039		1988 x 845 x 1039		1988 x 992 x 1039		1988 x 1139 x 1039			
lnn	er dimensions H x W x D (mm)	1167 x 447 x 713		1167 x 590 x 713		1167 x 740 x 713		1167 x 887 x 713			
Net	t weight with standard equipment (kg)	206		228		251		273			
	Supply voltage (V)	220-240	115-127	220-240	115-127	220-240	115-127	220-240	115-127		
	Frequency (Hz)	50 / 60	60	50 / 60	60	50 / 60	60	50 / 60	60		
y Sai	Power (W)	300	300	300	300	300	300	300	300		
	Energy consumption (kWh/24h)	1.50 / 1.40	1.40	1.50 / 1.50	1.40	1.50 / 1.60	1.40	1.60 / 1.60	1.40		
Ē	Heat emission (Kcal/h)	33	30	36	30	33	33	38	33		
	Compressor running time (%)	25	25	27	25	25	27	27	27		
	Noise level (dB(A)) (at 1m height & 1m distance)	46 / 49	52	47 / 51	50	50 / 51	46	47 / 47	46		



Specifications

















P55-130 & PF260

P290-380

P400-500-700-900

Regulation se	ensor	NTC, at 0°C ± 0	0.3°C	PT1000, at 0°C ± 0.1°C					
Display senso in reference body corning 200-5CST	/ / 100 ml DOW	NTC, at 0°C ± 0).3°C	PT1000, at 0°C ± 0.1°C					
	function time of the control system failure	12 V - 2.3 AH /	⁷ 48h	12 V - 7 AH / 48h					
Relative hum	nidity at +32°C	≤ 75%							
Door insulati	ion (polyurethane)	82 mm PU with 4-Layer PN	IMA organic glass	65-82 mm PU with 5-Layer PMMA organic glass					
Cabinet insul	lation (polyurethane)	60-65 mm F	PU	77-82 mm PU					
	Inner body	Polypropylene (copo	olymer UV)	Stainless steel (1.4301)					
	Outer body	Polypropylene (copo	olymer UV)	DX51D + Z100 coated					
Naterial	Inner door	Polypropylene (copolymer UV) Aluminium		Stainless steel (1.4301)					
nateriai	Outer door	Polypropylene (copo	olymer UV)	DX51D + Z100 coated					
	Drawer	Polycarbonate, tra	nsparent	Stainless steel (1.4301)					
	Shelf		Wire DIN177	7, PA11 coated					
European Me	edical Device Regulation		MDR (EU) 2017/745, C	ass (models: 220-240 V - 50/60 Hz)					
MC directive	e	2014 / 30 / EU (models: 220-240 V - 50/60 Hz)							
ow voltage (directive	2014 / 35 / EU (models: 220-240 V - 50/60 Hz)							
DA regulatio	on Medical Device	21CFR Part 862.2050, Class I (models: 115-127 V - 60 Hz)							
CSAus certif	ficate		Yes (models:	115-127 V - 60 Hz)					
GMP - Clean	room classification	Class ISO 6 / EC	GMP B	Class ISO 5 / EC GMP A					










ORGANIC

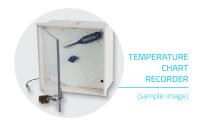

Equipment General

		P55	P130	P290	P380	PF260	P400	P500	P700	P900
B Medical	Digital display	•	•	•	•	2	-	-	-	-
Systems Electronics	7" full touchscreen display	-	-	-	-	-	•	•	•	•
Ergonomic han open/close - heavy d	ndle duty door lock mechanism	-	-	-	-	-	•	•	•	•
Organic glass door for quick check / pre-selection of the refrigerator's content		•	•	•	•	•	•	•	•	•
Internal LED lighting		•	•	•	•	•	•	•	•	•
°B Connected -	Monitoring Software	$\overline{\bigcirc}$	<u> </u>	$\overline{\bigcirc}$	\bigcirc	\bigcirc	\bigcirc	\bigcirc	\bigcirc	$\overline{\bigcirc}$
DCU - Data Communication Unit (in combination with °B Connected)		0	0	0	0	0	-	-	-	-
Ethernet interfo	ace	-	-	-	-	-	•	•	•	•
Temperature ch	hart recorder e form of a circular chart recorder)	0	0	0	0	0	0	0	0	0
Ambient tempe	erature sensor	-	-	-	-	-	0	0	0	0
Potential-free and a case of system fail		•	•	•	•	•	•	•	•	•
Integrated port (installed by custom	t for external sensor ^{ner)}	•	•	•	•	2	•	•	•	•
Rollers		-	2	• 2	2	2	-	-	-	-
Smooth castors with stabilizers		-	-	-	-	-	• 4	• 4	• 4	• 4
Door hinge Rigi	ht Left	•10	• 10	• 0	• 0	• 0	• 10	• 10	• 0	• 0









2 versions:





Equipment Internal storage

		P55	P130	P290	P380	PF260	P400	P500	P700	P900
Drawer with From	nt Cover	<u> </u>	<u> </u>	<u></u>	<u></u>	● 2 ○1	-	-	-	-
Drawer without Front Cover		<u></u>	○ ≤3	<u></u>	<u></u>	<u></u>	-	-	-	-
Bottom Drawer		1	1	1	1	2	-	-	-	-
Wire Shelf		• 1	• 2	5	• 7	• 2	● 2 ○≤3	● 2 ○≤3	● 2 ○≤3	● 2 ○≤3
ST-Drawer with Front Cover		-	-	-	-	-	3 ○≤2	3 ○≤2	3 ○≤2	● 3 ○≤2
ST-Drawer without Front Cover		-	-	-	-	-	○ ≤ 13	○ ≤ 13	○ ≤ 13	○ ≤ 13
Accessories										
	35 mm or H 100 mm) ndling and storage management	<u></u>	<u></u>	○ ≤5	○ ≤6	<u></u>	-	-	-	-
S-Rack (H 100 x 1	W 113 x D 638 mm)	-	-	-	-	-			$\bigcirc \le 25$ (max. per unit)	
CT D /	with Wire Shelf	-	-	-	-	-				
ST-Rack (H 95 x W 134	with ST-Drawer	-	-	-	-	-				
x D 648 mm)	with ST-Shelf	-	-	-	-	-				







S-RACK



Full functionalities at a glance

B Medical Systems | Electronics

PRECISION LINE | Multifunctional electronics with digital display and easy access, in the door handle; compatible with °B Connected monitoring solution.



• Key power switch • O = OFF / I = ON

- To switch the unit on, turn the key switch to position "I". The green power LED lights up
- After the self-check, the temperature inside the device is displayed




O Digital display

During normal operation the display shows:

- Left-hand side: current time and internal temperature displayed in °C (Celsius) or °F (Fahrenheit) in increments of 0.1°
 - Right-hand side: the maximum and minimum inside temperature



Function buttons

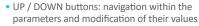



 ENTER button: use the ENTER-key to confirm the actual selection. This applies to the parameter value. The change of a parameter only becomes effective when confirmed



 MUTE / BACK button: in case of an alarm, the acoustic alarm can be muted for a specified time. In Parameter-mode the same key acts as "BACK-button" to go one step back



 LIGHT button: allows to switch on or off the interior light of the device without opening the door





When an alarm occurs, the red alarm LED lights up and an acoustic signal sounds. The corresponding alarm message appears on the display with the inside measured temperature. If several alarms occur simultaneously, the messages will be shown alternately.



A USB port is available for data export only. Measured values of the display sensor are stored for at least 30 days (ring buffer) together with a time stamp.



HAL

3.8°C

The new B Medical Systems Electronics

offers a wide range of adjustment and diagnostic functionalities as well as additional protection / warning operations (via external alarm operations, histories and individual display signals).

PREMIUM LINE | Multifunction electronics with 7" full touchscreen display, integrated at optimal level in the door handle with pre-installed connection allowing exclusive °B Connected monitoring functionalities.



Power switch OFF / ON

- To switch on press the ON/OFF button for 1 second. The green LED will light up
 - · Switch off protected by password. The door will automatically unlock before powering down



Touchscreen 7" full display

During normal operation the display shows:

- At the top: current voltage, current time, and login status (user level)
- Left-hand side: temperature of the regulation sensor (displaying this temperature can be disabled through the settings) and internal temperature displayed in °C (Celsius) or °F (Fahrenheit) in increments of 0.1°
- Right-hand side: touch buttons to access door unlock, clean, menu, alarm mute and message functions



O USB port & SD card slot

Acoustic and visual alarm signal

When an alarm occurs, the red alarm LED lights up and an acoustic signal sounds. The corresponding alarm message appears on the display together with the inside measured temperature. If several alarms occur simultaneously, the messages will be shown alternately.



Keycard reader

Login / Unlock using an NFC card

Touch buttons

- DOOR UNLOCK button: unlocks the door of the device
- CLEAN button: the cleaning button allows the user to disable the touch capability of the screen for a short period of time in order to clean the screen. This button is replaced by the logout button when user is logged in
- MENU button: allows access to the menu screen
- ALARM MUTE button: deactivates the acoustic alarm for a predefined period of time
- ACTIVE ALARMS AND WARNINGS button: during a warning or alarm situation, the color of this button and the corresponding level will change. Information on an active alarm and warning is accessible by pressing the message button













Optimum control and protection

B Medical Systems | °B Connected & DCU



B CONNECTED | MONITORING SOFTWARE

Universal software for the monitoring of refrigeration devices, including the acquisition, recording and visualization of temperature data.



- Unique monitoring software for the full range of Blood Management Solutions and Medical Refrigeration products
- Web-based interfaces for computers and mobile devices
- Modern design for simple and intuitive use
- Graphic display of temperature curves
- Integrated event and activity history of appliances' components
- Data recording on centralized database for long-term archiving
- Easy setting of specific alarm, via email or SMS alerts
- Generation of reports compiling crucial data and activities
- Temperature and detailed device data export for third-party software
- Important cost advantage compared to a traditional circular chart recorder and its spare parts
- REST API to access raw data directly from the database in read-only mode

KEY BENEFITS:

- Unique interface for the monitoring of the full range of refrigeration devices
- Centralized database providing data access to entire customer network







settings, real-time clock, language, firmware

version, RS485 address, IP address

- > Complete & legally safe documentation of temperature data
- > Comprehensive applications and diagnostic functionalities







DCU | DATA COMMUNICATION UNIT (PRECISION LINE)

Hardware module monitoring all operating conditions and passing them through to a central database – via local network.



- Interface connection of B Medical Systems appliances to an existing network
- Graphical user interface displaying temperature and configurating devices and alarms easily
- Direct connection to Ethernet and serial bus RS485
- Digital in/out (programmable and customer-specific use)
- Recording and storing appliance-relevant data
- Integrated USB port allowing to export archived data
- The DCU combined with the °B Connected software replaces the paper temperature recorder
- All data are recorded and saved in the internal storage of the DCU and backed up in °B Connected database if connected
- Several additional self-sufficient temperature sensors (up to 4 PT1000) may be connected
- Humidity sensor input (4-20 mA)

KEY BENEFITS:

- Central system collecting relevant temperature data of the appliances and their respective operating conditions
- A number of connection abilities allowing flexible upgrades for individual projects





around the world

Reliable solutions for safe vaccination



State-of-the-art technology for the exacting needs of the medical world

Our Global Expertise





After Sales support and service

We strive to provide you with the highest standards of service; not only through our selected distributors and partners for all your maintenance and service but also our second line trouble shooting and after sales service. This factory-based group of engineers is there to help our partners and yourself to get the best solution for your cold storage needs.

Safe global blood management: from collection to transfusion, transportation, processing and storage



SAVING LIVES THROUGH RELIABLE AND INNOVATIVE TECHNOLOGY

B Medical Systems (formerly Dometic Medical Systems) has more than 40 years' experience in the medical refrigeration sector.

The company, formerly known as Electrolux Medical Systems, was founded in 1979 when the World Health Organization approached Electrolux in Vianden, Luxembourg, to create a solution for the safe storage and transport of vaccines around the world. In 2001, Electrolux Medical Systems became part of the Dometic Group, and was renamed Dometic Medical Systems. Having established a legitimate reputation in the medical equipment industry, the company has also become a global leader in vaccine cold chain.

B Medical Systems S.à r.l.

17, op der Hei L - 9809 Hosingen, Luxembourg

Tel.: (+352) 92 07 31-1 Fax: (+352) 92 07 31-300 info@bmedicalsystems.com















Since 2019 B Medical
Systems has been committed
to the UN Global Compact
corporate responsibility
initiative and its principles
in the areas of human rights,
labour, the environment
and anti-corruption.

Luxembourg, in the heart of Europe

